

Wound Care Protocol using L-Mesitran™ honey dressings

Antibacterial - Remove malodour - Stimulate cellgrowth (fibroblast/keratinocytes) - Reduce scarring

Hvdro / Border

SUPERFICIAL WOUNDS DEEP WOUNDS


CHRONIC AND ACUTE WOUNDS



The wound is RED

- protect
- moist environment
- bacterial balance
- stimulate epithelisation


Clean the wound first (e.g. saline) Wet Soft Net Moist Tulle / Soft Soft Soft Ory Tulle / Soft Soft

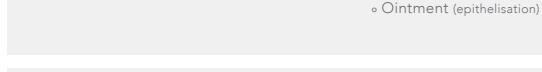



The wound is YELLOW


- $\circ \; \mathsf{debride}$
- o remove bacteriae
- moist environment
- o stimulate healing

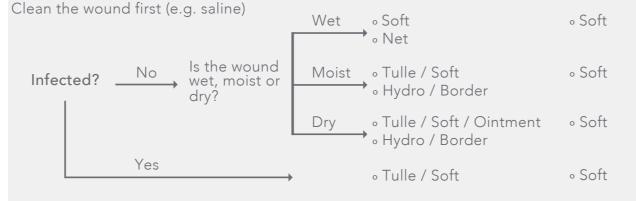



The wound is BLACK

- cleanse
- remove necrosisremove bacteriae

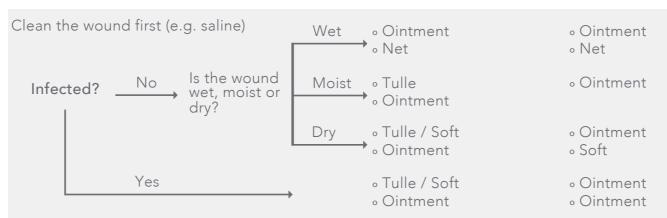

POST-OPERATIVE WOUNDS



- © Triticum 2015 www l-mesitran com D066 rev

L-Mesitran Ointment:

- Wound ointment with medical honey, Medilan, Aloë, Calendula, ZnO and anti-oxidants vitamins C and E
- Indicated for the removal of necrosis and debris (cleaning function)
- Cover with alginate, hydrofiber, foam or compress
- Replace the ointment after 1-3 days



L-Mesitran So

- Woundgel with medical honey, Medilan, PEG and anti-oxidants vitamins C and E
- Cover with foam, alginate, hydrofiber or compress
- Indicated for all superficial and deep wounds (antibacterial and stimulates wound healing)
- Replace the gel after 1-3 days



L-Mesitran Tul

- Hydro-active synthetic sheet with L-Mesitran Soft
- Does not adhere to the wound
- Indicated for wounds without or moderate exudates
- Cover with absorbent dressing, fixate with regular gauze
- Replace the dressing after 3-5 days



L-Mesitran Hydro

- Hydro-active dressing with medical honey, polymers and water
- Indicated for superficial wounds
- Self adhering, can be fixated with foil, wide fixation dressing or (adhesive) tape
- Replace the dressing after 3-5 days



L-Mesitran Borde

- Hydro-active dressing with medical honey, polymers and water with an adhesive border
- Indicated for superficial wounds
- Self-adhesive, showering or swimming is possible
- Replace the dressing after 3-5 days



Soft

I Mositran Not

- Hydro-active wound contact later with medical honey, polymers and water
- Indicated for wounds with moderate to heavy exudates
- Cover with absorbent dressings or compresses and fixate with regular adhesive gauzes
- Replace the dressing after 3-5 days